Hoof Trim Chute Instructions & Tips





Eastern Location

15242 - 460th Street Clearbrook, MN 56634 Phone: 218-776-3188

or 800-474-6016


E-mail: dahlberg@realtuff.com

Western Location

28727 US Highway 12 Mobridge, SD 57601 Phone: 605-845-3456

or 800-881-3457


E-mail: realtuff@westriv.com

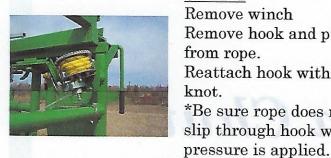

STEP 1 Remove winch Remove hook and pulley from rope. Reattach hook with tight knot. *Be sure rope does not slip through hook when



STEP 7 Bring winch from inside HTC to outside arm as shown.

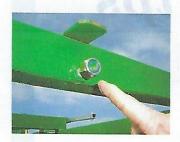

STEP 2 Remove nut.



STEP 8 Reattach winch to arm.



STEP 3 Pulley goes on front bolt.



STEP 9 Lock arm in working position with clip pin.



STEP 4 Second bolt for second belt kit which is optional.



STEP 10 Remove winch which operates back butt strap.



STEP 5 Winch, pulley, and rope in place.



STEP 11 Reattach pulley as shown.



STEP 6 Remove pulley from back leg lift arm.



STEP 12 Front winches fit into working notch.



STEP 13

Winch rotates out to open wide area for doctoring or to create more working area for cow.



STEP 19

Tabs on rear of chute for connecting palpation cage or panels.



STEP 14

Winch folds into HTC so it's out of the way for traffic passing close by chute. Also needs to be folded in if chute is placed on a chute cart.



STEP 20

When using one belt, hook the chain into middle hook. Front and rear hooks are used if the unit is used with a second belt kit.



STEP 15

Back leg lift in closest position.



STEP 21

Both side doors easily latch in "up" position for full and easy access to work on animal.



STEP 16

Back leg lift in center position which is the position that will be used the majority of the time.



STEP 22

Remove "one" pin on top of pipe of fold up door.



STEP 17

Back leg lift in farthest position.



STEP 23

Door is completely removable for doing a DA or other doctoring where big exposure is needed.



STEP 18

Locate all grease zerks and grease when needed.



STEP 24

Wing nut adjusts to limit opening of self catching headgate.



STEP 25

Pins remove top and bottom of each side of headgate so width of opening can be adjusted.



STEP 28

Back butt strap in place to push cow forward into headgate and hold them tight while working on cow.



STEP 26

Hook can be on right or left side on the top of the headgate for holding IV bottle.



STEP 29

Back lift arm completely recessed out of the way to give cow full access to enter HTC.



STEP 27

Belly belt in place.



STEP 30

- Spring makes headgate reset automatically to previous position when cow walks through.

- Lock can be in position when headgate is wide open which will keep headgate locked open.

Hoof Trim Chute Use Tips

To trim hooves or work the feet of your animal, here are some tips:

- 1) Draw strap very tightly on the butt of your animal.
- 2) Secure big belly belt in place and tighten until there is very little pressure on front feet.
- 3) Lift one of rear legs first.
- 4) Keep one of the back legs lifted while you work on the front feet.
- 5) Lift either of the front legs
- 6) The front leg lift mechanism works better if it is raised to the top of its slide and the telescoping shaft is pulled out 2-3 inches. Tighten nuts and adjustment bolts when you have your desired position.
- 7) Working the front leg is more difficult so practice and practice.

HTC Rope Dimensions

Length
6 or 7' each
11'
14'
18'
11'

A HTC has either 1 belly belt or 2 belly belts. Each chute will have 4 ropes total. *It will also have a butt strap.*

A Double Belly Belt Kit has the following:

- 1) One belly belt which will be the second belly belt
- 2) Hook
- 3) Pulley wheel
- 4) 18' Rope